


BioSign® Strep A-Flip

Immunoassay for the detection of Group A streptococcus antigen directly from throat swab specimens

ALL NEW FLIP TEST DESIGN!!!

BioSign® Strep A Flip Please read the instructions carefully before performing the test.


TEST DEVICE

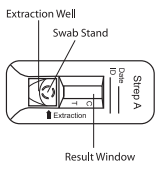

PROCEDURE

- 1 Add 4 drops of Reagent A into Extraction Well.
- 2 Add 4 drops of Reagent B into Extraction Well.
- 3 Place the specimen swab in Swab Stand in Extraction Well. Hold the device with one hand and the swab with the other. **While pressing down on the swab**, spin it **in one direction** about 5 times to mix the specimen.
- 4 Incubate 2-5 minutes with the swab in Extraction Well.
- 5 Hold the device with one hand and the swab with the other. **While pressing down on the swab**, spin it **in one direction** about 5 times to mix the specimen. Remove the swab and discard.
- 6 Raise the device upright.
- 7 Tap the device on a flat surface to ensure that the liquid flows into the hole.
- 8 Slowly lay the device back down onto flat surface.
- 9 Read test results after 5 minutes but not after 10 Minutes.

INTERPRETATION OF RESULTS

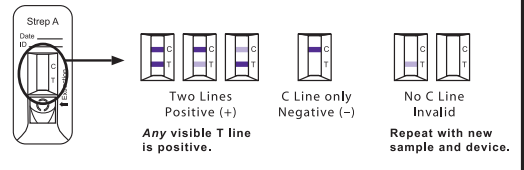

Two Lines Positive (+)
Any visible T line is positive.

C Line only Negative (-)
Repeat with new sample and device.

No C Line Invalid
Repeat with new sample and device.



CLIA Waived

Easy to use flip step

Quick and accurate detection of Group A Strep with walk away convenience

Results in 5-10 minutes

Sensitivity of 96.2% and Specificity of 98.7%

For more information about LifeSign Strep A Kits, or to place an order, contact:



ORDERING INFORMATION

PRODUCT	KIT SIZE	ITEM #	CPT#
BioSign® Strep A - Flip	25	BSP-182	87880QW
Other Available Products:			
Status® AccuStrep A	25	34025	87880
BioStrep® Strep A	30	BSP-185	87880QW

LifeSign and BioSign are registered trademarks of Princeton BioMeditech Corporation. Printed in USA. All rights reserved.



For More Information Contact:

The Infolab Distribution Center Nearest You

Clarksdale, MS**Toll Free: 1-800-647-8222****Phone: (662) 627-2283****Fax: (662) 627-1913****Email:****infolabms@infolabinc.com****Fort Oglethorpe, GA****Toll Free: 1-800-241-8953****Phone: (706) 861-5046****Fax: (706) 866-5234****Email:****infolabga@infolabinc.com****Greensboro, NC****Toll Free: 1-800-782-9700****Phone: (336) 272-2874****Fax: (336) 274-9473****Email:****infolabnc@infolabinc.com****Round Rock, TX****Toll Free: 1-800- 322-3882****Phone: (512) 255-7994****Fax: (512) 244-9659****Email:****infolabtx@infolabinc.com****Ocala, FL****Toll Free: 1-800- 734-7584****Phone: (352) 237-9654****Fax: (352) 237-9733****Email:****infolabfl@infolabinc.com****Zionsville, IN****Toll Free: (866) 232-7096****Phone: (317) 873-4400****Fax: (317) 873-4443****Email:****infolabnc@infolabinc.com**