


Strep A

Immunoassay for the detection of Group A streptococcal antigen directly from throat swab specimens

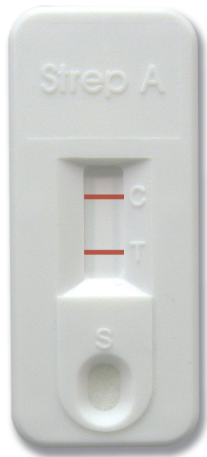

Status®
Accustrep A



BioStrep® A
Strip

Status® AccuStrep A (Non-Waived)

Features

- Quick and accurate detection of Group A Strep
- Simple 2-step procedure
- Flexible extraction and reaction times
- Results in 5-10 minutes
- Sensitivity of 96.2% and specificity of 98.7%

Test Procedure*

1. Dispense 4 drops of Reagent A and 4 drops of Reagent B into the extraction tube and place the specimen swab in the extraction tube. Rotate the swab vigorously. Incubate at room temperature for 1-2 minutes.
2. Remove the swab while squeezing out the liquid.
3. Dispense 4 drops of extraction into the cassette sample well using the dropper provided.
4. Read results in 5-10 minutes
5. Positive Results: Reddish lines appear in both the Control (C) and the Test (T) area.
6. Negative Results: Only one colored line appears in the Control line area (C).

*See package insert for details

BioStrep® A (Waived)

Features

- Rapid and accurate diagnosis assures prompt treatment
- Simple procedure
- Compact strip size
- Results in 5-10 minutes
- Sensitivity of 96.2% & specificity of 98.7%
- Easy extraction for fast reaction times
- CLIA waived

Test Procedure*

1. Dispense 4 drops of Reagent A and 4 drops of Reagent B into the extraction tube and place the specimen swab in the extraction tube. Rotate the swab vigorously. Incubate at room temperature for 2-5 minutes.
2. Remove the swab while squeezing out the liquid and insert the test strip into the tube.
3. Read results in 5-10 minutes
4. Positive Results: Reddish lines appear in both the Control (C) and the Test (T) area.
5. Negative Results: Only one colored line appears in the Control line area (C).

*See package insert for details

Other Testing Solutions

- hCG
- Drugs of Abuse
- H.Pylori
- Cardiac Markers
- Mono

For more information about LifeSign Strep A kits, or to place an order, contact:



ORDERING INFORMATION

PRODUCT	KIT SIZE	ITEM #	CPT#
Status® AccuStrep A	25 tests	34025	87880
BioStrep® A	30 tests	BSP-185	87880QW

LifeSign, BioSign, and BioStrep are registered trademarks of Princeton BioMeditech Corporation. Printed in USA. All rights reserved.



For More Information Contact:

The Infolab Distribution Center Nearest You

Clarksdale, MS**Toll Free: 1-800-647-8222****Phone: (662) 627-2283****Fax: (662) 627-1913****Email:****infolabms@infolabinc.com****Fort Oglethorpe, GA****Toll Free: 1-800-241-8953****Phone: (706) 861-5046****Fax: (706) 866-5234****Email:****infolabga@infolabinc.com****Greensboro, NC****Toll Free: 1-800-782-9700****Phone: (336) 272-2874****Fax: (336) 274-9473****Email:****infolabnc@infolabinc.com****Round Rock, TX****Toll Free: 1-800- 322-3882****Phone: (512) 255-7994****Fax: (512) 244-9659****Email:****infolabtx@infolabinc.com****Ocala, FL****Toll Free: 1-800- 734-7584****Phone: (352) 237-9654****Fax: (352) 237-9733****Email:****infolabfl@infolabinc.com****Zionsville, IN****Toll Free: (866) 232-7096****Phone: (317) 873-4400****Fax: (317) 873-4443****Email:****infolabnc@infolabinc.com**